

Re-Order #	Description	Size	Weight	Unit
912449B	Quad Eight Spoke - Blue	15" x 15"	2.2 lbs	Each
912449G	Quad Eight Spoke - Green	15" x 15"	2.2 lbs	Each
912449C	Quad Eight Spoke - Clear	15" x 15"	2.2 lbs	Each
912449R	Quad Eight Spoke - Red	15" x 15"	2.2 lbs	Each
912449Y	Quad Eight Spoke - Yellow	15" x 15"	2.2 lbs	Each



Purpose

The brain seeks information primarily by directing individuals to look, listen, smell, touch and taste. It is important to note that the experience of any event is almost always multi-modal involving the sensory and motion systems (Freeman, 1991, Thelen & Smith, 1997). It has been found that therapeutic touch (Krieger, 1997) is effective in decreasing anxiety. These multi-modal products address sight and touch and are effective in reducing anxiety, boredom and also serve as a tool in training for eye-hand coordination and also improve finger strength and dexterity.

Easy to Use

These sensory pads should be placed on a surface convenient to the child. The individual should be encouraged to move the marbles around the shape or to trace their finger along the contour of the sensory pad. These pads may be warmed in warm water or cooled in the refrigerator to enhance recognition of heat or cold. Since the marble components present a possible choking hazard these products shouldn't be used for persons cognitively under (3) years old. In addition, these products must always be used under the direct supervision of a care giver.

Easy Care

These vinyl pads are heat sealed and can be cleaned with a mild detergent. Although the gel is non-toxic, medical care should be consulted if the gel is ingested. If a tear or rip is observed the unit should be discarded.

Caution

Possible choking hazard. Do not use for persons cognitively under three (3) years old.